InnoSense

Instruction Manual

InnoWhite[®]

Professional Dental Whitening System

EN

InnoWhite InnoSense reduces and prevents tooth sensitivity. Its effectiveness is clinically proven.

DESCRIPTION

InnoSense is a water-based gel with active ingredients against tooth sensitivity; hydroxyapatite and potassium nitrate. The mint aroma adds a pleasant taste. InnoSense remineralises the teeth, which makes the teeth stronger, significantly reduces sensitivity and the whiteness of the tooth is enhanced.

PACKAGING

Syringe, 3ml / 3.5 grams.

COMPOSITION

Key-ingredients: Hydroxyapatite 2%, Potassium nitrate 0.1%

INDICATIONS FOR USE

- Prevention and reduction of tooth sensitivity.
- After teeth whitening in dental practice and / or at home.
- After professional cleansing of the teeth.

CONTRA-INDICATIONS

Do not use if allergic to mint or one of the other ingredients.

HANDLING PROCEDURE

Clean teeth beforehand by brushing and flossing.

APPLICATION

WITHOUT TRAY:

Wash hands thoroughly with water and soap. Dispense a small amount of InnoWhite InnoSense on a clean finger and apply it on the upper and lower teeth. Leave the gel on for 10 minutes. Spit out any excess gel but do not rinse afterwards.

WITH TRAY:

Dispense a small amount of InnoWhite InnoSense across the entire tray. Place the tray in the mouth and move the tray a little from left to right to spread the gel on the teeth. Gently bite down on the tray and leave the tray with the gel in place for 10 minutes. Remove the tray and spit out any excess gel but do not rinse afterwards. Rinse tray with cold water.

- Always close the tube with the cap immediately after use.
- For best results: do not drink or eat for at least 30 minutes after application.
- Use InnoSense once a day or when needed. Please follow the instructions of your dental professional.

WARNINGS & FIRST AID INSTRUCTIONS

In case of contact with the eyes, flush thoroughly with plenty of water and seek medical attention. Do not use the product after expiry date.

STORAGE ADVICE

Store in a cool and dry place (frost-free) and protected from direct sunlight.

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ESSENTIAL INNOVATION PTE LTD